Operating Manual

# Ambu<sup>®</sup> Manikin Management Module

English.....	3
Česky.....	23
Dansk .....	44
Deutsch .....	65
Ελληνικά .....	88
Espanol.....	111
Suomi .....	133
Francais.....	154
Magyar.....	177
Italiano .....	199
日本語.....	222
Nederlands .....	244
Norsk .....	266
Polski.....	287
Portugues .....	309
Русский .....	331
Svenska.....	353
Turkce.....	374
中文 .....	395

# English

## Contents

1	Introduction.....	5
2	Requirements .....	5
2.1	Operating system and Hardware.....	5
2.2	Software .....	5
3	Symbols and Buttons .....	5
4	Getting started .....	7
5	Controlled session .....	7
5.1	New session .....	8
5.2	Session View .....	8
5.2.1	Manikin selection .....	8
5.2.2	Functional menus .....	9
5.2.3	Performance graphics .....	10
5.2.4	Physiological values .....	10
5.2.5	Event selection .....	10
5.3	Surveillance view .....	11
5.4	Training results .....	11
5.5	Training course.....	11
5.6	Database .....	12
5.6.1	Scenario .....	12
5.6.2	Algorithm .....	13
5.6.3	Modes.....	13
5.6.4	Sessions.....	14
5.6.5	Trainees .....	14
5.6.6	Events .....	14
5.6.7	ECG.....	14
5.6.8	Sound.....	14
5.7	Configuration .....	14
5.7.1	Display.....	15
5.7.2	Viewing mode .....	15
5.7.3	Debriefing .....	15
5.7.4	Print.....	16
5.7.5	Storage.....	16
6	Viewing mode .....	16
6.1	Graphical display in viewing mode.....	17
6.2	Line graph:.....	17
6.3	Bar graph:.....	17
6.4	Bubbles:.....	17
6.5	Patient Monitor:.....	18
7	Self-training.....	18
8	Manikin Configuration .....	19
8.1	Overview.....	19

8.2	Network settings .....	20
8.2.1	Single Manikin .....	20
8.2.2	Multi-Manikin .....	20
8.3	Introduction screen .....	21
8.4	Self-training .....	21
8.5	System .....	21
8.6	Administration .....	22

# Operating Manual

## 1 Introduction

The Ambu Manikin Management Module is browser-based, intuitive, modular software application used for the wireless control of the Ambu manikins, including the possibility for monitoring and documentation of the performance of trainees.

## 2 Requirements

### 2.1 Operating system and Hardware

The minimum requirements for the Ambu Manikin Management Module are:

Operating system	Version	CPU	RAM
Mac OS®	10.9	1,8 GHz	4 GB
iOS®	6.1	1,4 GHz	1 GB
Android™	4.1	2,3 GHz	2 GB
Microsoft® Windows®	7	1,8 GHz	1 GB

### 2.2 Software

To access the Ambu Manikin Management Module a web browser is needed. The minimum requirements for suitable web browsers are:

Web browser	Version
Internet Explorer®	10
Safari®	7
Chrome™	44
Firefox®	37

The Ambu Manikin Management Module will automatically rescale the view to the display size of the device.

For optimal overview, a minimum display size of 9.7" is recommended.

Note. The Ambu Manikin Management Module uses Java™ scripts. The user has to make sure that Java scripts are activated and blockers are disabled. If you are unable to change the settings, consult your administrator.

## 3 Symbols and Buttons

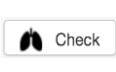
The following section shows an overview the symbols and buttons, and gives further details on their function.

Note: Availability of symbols or buttons is dependent on the manikin model.


Application menu

-  Start new session
-  Open training course window
-  Open team debriefing
-  Open session view window
-  Open database window
-  Open surveillance view window
-  Open configuration window

Session

-  Open sound menu<sup>1</sup>
-  Play / start
-  Open switch position menu
-  Open ECG menu<sup>1,2</sup>
-  Pause
-  Exit
-  Open physiological parameter menu<sup>1</sup>
-  Running on AC power
-  Battery charging
-  Battery status
-  Fast backward
-  Current time
-  Fast forward
-  Backward
-  Forward
-  Forward
-  Check
-  Event button (can have different pictures)
-  Backward
-  Decrease value
-  Checkmark enabled
-  Open drop down menu
-  Increase value
-  Checkmark disabled
-  Open 12 lead ECG<sup>1</sup>
-  Results / scores
-  Print
-  Delete
-  save

## Configuration/database

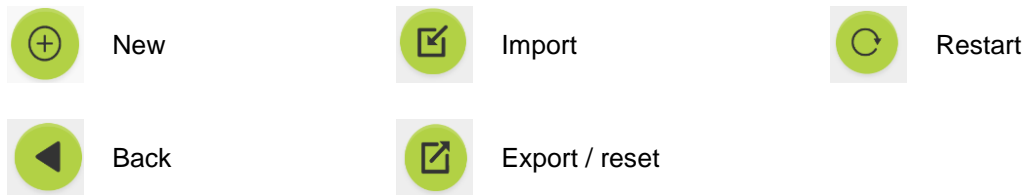
<sup>1</sup> AmbuMan Advanced

<sup>2</sup> AmbuMan Defib

## 4 Getting started

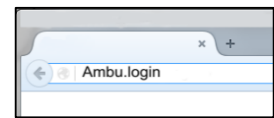To get started with the Ambu Manikin Management Module follow the steps below:

1. Turn on the manikin by briefly depressing the power button located in the lower torso.
2. Connect your computing device to the manikin wireless (WLAN) or using a network cable.  
When using WLAN connect to the wireless network “AmbuW”, no password is required.
3. Open one of the web browsers described in Section 2.2
4. Enter “http://Ambu.login” into the address field of the web browser.  
If the web browser does not accept “http://Ambu.login” type in the IP address “192.168.1.1”
5. The Ambu Manikin Management Module start screen is loaded and you can choose:
  - “Controlled session” (for further details see section 5)
  - “Viewing Mode” (for further details see section 6)
  - “Self-training” (for further details see section 7)
6. Click on the button of the desired mode to start your session.


## 5 Controlled session

The controlled session is intended to be used by a training instructor.

The page is split into the application menu on the left side (①) and the functional area on the right side (②).

The application menu will automatically rescale to allow optimal view of the functional area. If the computing device is turned to a vertical view, the menu will only display the icons.

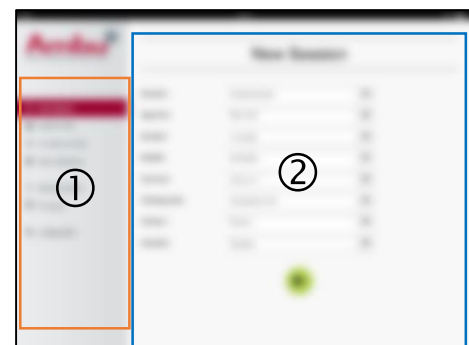Further details are explained in the section 5.1 to 5.7.

## 5.2.2 Functional menus ②

The functional menus allow you to change the behaviour of the manikin and to adjust the scenario during the running session.

### 5.2.2.1 Sound menu

The sound menu allows you to play sound through the manikin.

- Open the sound menu by clicking on the sound menu button
- Click on the drop down button to select the required sound.
- Once selected press the play button to play the sound through the manikin.
- To stop the sound click on the pause button.
- To change the volume of the played sound use the “+” or “-” buttons.

The volume change will take place automatically. It is not necessary to click on the play button again,

To hide the sound menu, click on the sound menu button.

Note. The frequency of the sound files are not directly linked to the physiological values. If the breathing frequency is manually changed, the played sound might not match the breathing frequency. In this case, the sound file with the correct frequency should be selected.

### 5.2.2.2 ECG menu

The ECG menu allows you to play or change ECG signals generated in the manikin. Open the ECG menu by clicking on the ECG menu button

Click on the drop down button to select the required ECG signal.

Once selected press the play button to activate the ECG signal in the manikin.

To change the frequency or the amplitude of the ECG signal use the “+” or “-” button.

Note. Some ECG simulations can physiologically be wrong by changing the frequency. In these cases, select the correct ECG signal from the drop down menu. (i.e. Art. flutter).

Defibrillation/pacing:

If required, the ECG signal can be automatically changed after defibrillation:

- Select the new ECG signal from the drop down menu.
- Activate the change by clicking on the play button.
- To overwrite the automatic change of ECG signal after defibrillation/pacing activate the check mark in the selection “no defib effect”.

Carotid pulse:

If an ECG signal with pulse is selected, the pulse will automatically be generated on the manikin.

To deactivate, remove the check mark from the selection.

ECG artefacts:

The system allows the artefacts “muscle tremor” or “loose electrode” to be added to the ECG signal.

To activate/deactivate an artefact, set or remove the check mark from the selection.

To hide the ECG menu click on the ECG menu button

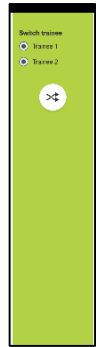
### 5.2.2.3 Switch user menu

The Ambu Manikin Management Module will evaluate the performance of the trainees during the session.

In order to get an individual evaluation for each trainee, the instructor needs to switch the position of the trainees during the session:

- Open the Switch User menu and click on the switch menu button.
- Select the 2 trainees that will switch positions.
- Activate the change by clicking on the “play” button.

To hide the switch trainee menu click on the Switch menu button

### 5.2.2.4 Physiological parameter menu

The manikin can display the physiological parameters in the instructor view, the viewing mode or the patient monitor.

The physiological parameters are linked to the selected ECG.

- To change the physiological values, open the physiological menu by clicking on the menu button.
- To change the values use the “+” or “-“ button.

To hide the physiological value menu click on the menu button.

### 5.2.2.5 Play/Pause session

To start the session click on the “play” button.

To pause the session click on the “pause” button.

### 5.2.2.6 Exit session

To cancel, exit or to stop a session before the selected duration has been reached, click on the “exit” button.

### 5.2.3 Performance graph ③

The performance graph shows the performance achieved by the trainees.

The top graph represents ventilation, the middle ECG and the bottom graph represents compression.

If a compression with a wrong hand position or a stomach inflation is performed, an “X” will be displayed on the graphs.

The type of display can be changed in the configuration, see section 5.7.1.

If manual events are selected, the icon will be displayed in the graph for the selected trainee.

To scroll through the graphs, the forward and backward buttons can be used.

### 5.2.4 Physiological values ④

The physiological values selected are displayed in the box at the bottom of the screen.

In addition to the physiological values, the compression and ventilation frequency performed by the trainees are displayed in real time.

### 5.2.5 Event selection ⑤

The system allows the instructor to add events to the session that the manikin cannot automatically detect.

To add the required event, click on the event button for the trainee who has performed the action.

To scroll through the events, click on the scroll button on the left or right side.

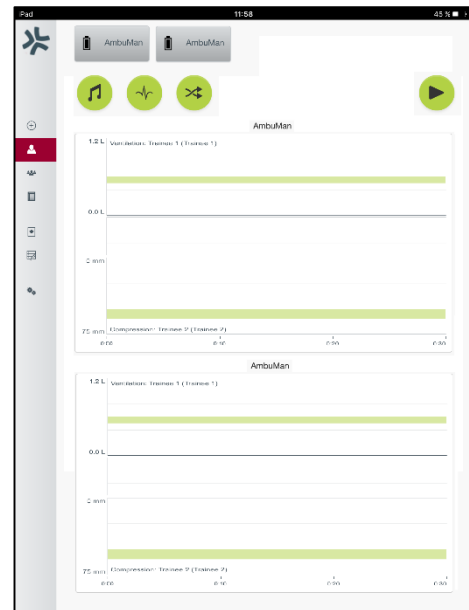
### 5.3 Surveillance view

When several manikins are used in parallel, the Surveillance View can be used to view the performance of all manikins.

To make changes to as described in section 5.2:

- Select the required manikins by clicking on the manikin buttons on the top.
- Click on the required functional button

The changes will automatically be done on all selected manikins.

### 5.4 Training results

Once the session is finished, the training results are available.

The training result page gives an overview of the selected session and the performance achieved.

The detailed performance for each trainee is shown in the table.

The results can be printed, saved or deleted by clicking on the corresponding button.

Note. The values shown in the training results can be changed in the configuration. See section 5.7.3.

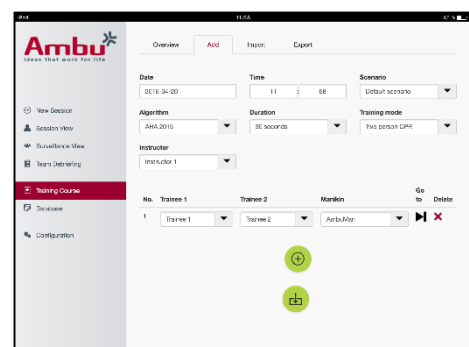
### 5.5 Training course

A Training Course allows an instructor to configure and prepare a training in advance, in order to save time during the training.

In the training course, all parameters are saved, so that the instructor can start the session without further work.

To create a new course:

- Click on the “Add” tab.
- Select the training parameters from the drop down menu.
- Select the trainees from the dropdown menu or type the names into the text field.
- Click on the “+” button to create a new training session.
- When all names are entered click on the “save” button.

In the overview tab, the training course will be available and the sessions can be started by clicking on the “play button”

A training course can be created without the use of a manikin. In order to do this a csv template can be exported from the manikin, the course created with it, then imported back to the manikin.

Note. To create a training course without the manikin the ID's of the different parameters have to be known. The ID's of all parameters can be displayed as described in the section 5.7.1.

## 5.6 Database

The database section is designed the same way for all entries.

The overview page gives an overview of the entry loaded into the manikin.

The Add page allows the addition of a new entry, or the modification of an existing entry, to the manikin.

To add an entry follow the display on the screen and click on the save button when finished.

The import page allows the manikin to import entries from the computing device.

To import a file to the manikin:

- Click on the browse button and select the file on the computing device.
- Click on the import button to upload the entry to the manikin.

The export page allows exporting single or multiple entries to the computing device.

To export a file from the manikin:

- Select the entry or entries to be exported
- Click on the export button.
- Select the folder on the computing device to store the entry on the device.

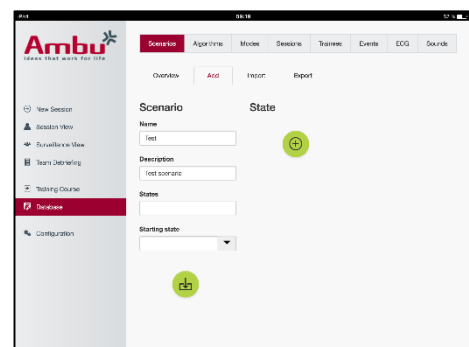
### 5.6.1 Scenario

A scenario is a predefined algorithm that allows the manikin to automatically react to the actions performed.

A scenario consists of a status the manikin is currently running, an action that needs to be performed and a future status the manikin shall change to after the action.




To create a new scenario the following steps are necessary.

1. Prepare the scenario and decide on sequence of the actions to be done on the manikin.
2. Enter the name and the description of the scenario.
3. Click on the “add” button to insert a state.
4. Enter the name of the state, i.e. “Start”.

5. Select the setting for this state.
6. Click on the “add” button to add the new created state to the scenario.
7. Now create the remaining states the scenario shall have.
8. Once all states are finished, continue to add the required actions to the single states.
9. Select the first state and click on the “add” action button.
10. Type in the name of the action for better overview.
11. Select the trigger (the action) from the drop down menu and set the trigger value
12. Finally select the new state.
13. Now create the remaining actions for the different states.
14. Finally select the starting state from the dropdown menu and click on the “save” button to upload the scenario to the manikin.

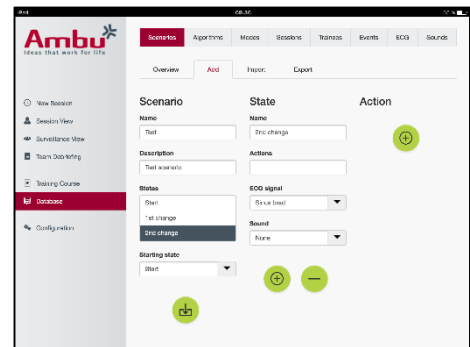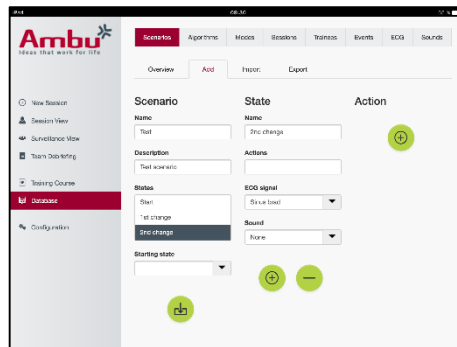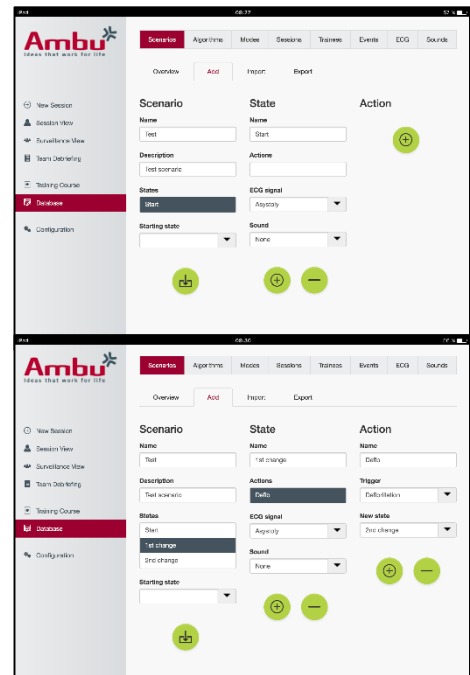A scenario can be created without the use of a manikin. In order to do this, a csv template can be exported from the manikin, the scenario created with it, then imported back to the manikin.

Note. To create a scenario without the manikin the ID's of the different parameters have to be known. The ID's of all parameters can be displayed as described in the section 5.7.1.

## 5.6.2 Algorithm

The algorithm tab is the basis of the evaluation as it defines the correct values for the actions to be done.

The predefined algorithms are in conformance to the latest CPR guidelines 2015.

When creating new algorithms fill in the fields and click on the save button.

## 5.6.3 Modes

The mode tab describes the kind of training to be performed and how many trainees are involved in the training.

#### **5.6.4 Sessions**

The session tab gives an overview of the saved sessions on the manikin.

The sessions can be re-opened to view the performance.

Note: The content of a saved session cannot be changed.

#### **5.6.5 Trainees**

The trainee tab gives an overview of the saved trainees on the manikin.

When adding a trainee to the database, it can be selected if the trainee is also an instructor.

#### **5.6.6 Events**

The event tab gives an overview of the saved events on the manikin.

When adding or editing an event the icon to be displayed can be selected.

If the required icon is not available, a new icon can be uploaded to the manikin.

When ready, click on the “save” button

Icons need to be in PNG format with a size of 20x20 pixels.

#### **5.6.7 ECG**

The ECG tab gives an overview of the saved ECG's on the manikin.

To create a new ECG signal, the template can be downloaded from the manikins.

The template contains a description of the inputs needed.

In the template, each line corresponds to 0,02 seconds. Each data point corresponds to 0,172 mV ECG signal amplitude.

Existing ECG signals can be modified by exporting the signal from the manikin, editing the signal, then importing it back to the manikin.

#### **5.6.8 Sound**

The sound tab give an overview of the saved sounds on the manikin.

New sounds can be uploaded to the manikins.

Sounds files need to be in WAV (8MHz, mono, 16-Bit, libsndfile: waveform audio, a-Law 8Bit PCM (ITU-T G.711)) format, with a maximum size of 3 MB.

### **5.7 Configuration**

The format information is displayed in a session can be configured according to personal needs.

Note: The configuration cannot be changed while a session is active.

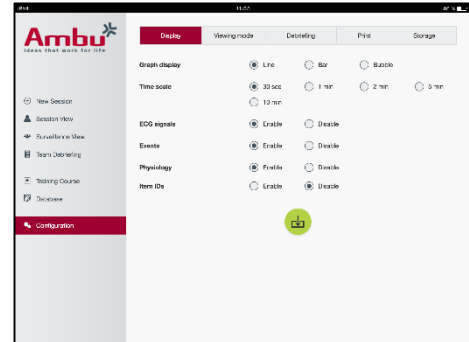
Some functions or buttons are only available on certain manikin models

The network configuration cannot be changed in this section. For modifying network settings, see section 8.

### 5.7.1 Display

The display view is what the instructor sees on the computing device. The view can be configured to show the relevant parameters according to personal needs.

To change the setting, enable or disable the relevant checkmark.


When finished click on the “save” button to store the settings.

### 5.7.2 Viewing mode

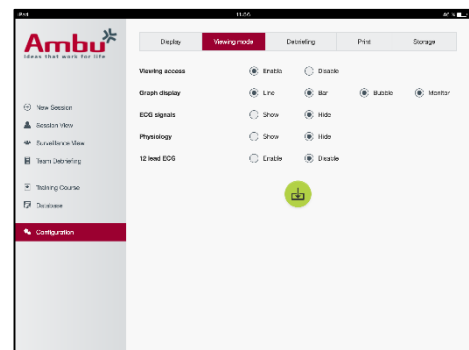The viewing mode is a secondary display view that can be used by the trainees as described in section 6.

The instructor can select if the viewing shall be available in general and what information will be displayed.

To change the setting, enable or disable the relevant checkmark.

When finished click on the “save” button to store the settings.

### 5.7.3 Debriefing

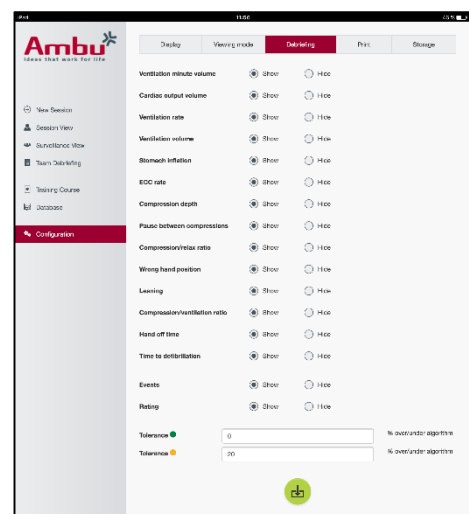
The debriefing section shows all the parameters that can be displayed for evaluation of the session.

The instructor can select which parameters are important for the evaluation according to their personal needs.

In addition, the instructor can change the tolerances of the selected algorithm, to allow a more detailed view of how much the performance is away from the recommendations.

Changing the tolerances will change the colouring of the parameters in the debriefing, bar and bubble display.

To change the setting enable or disable the relevant checkmark.

Note. Changing of the tolerances does not have an influence on the automatic rating of the session.

When finished click on the “save” button to store the settings.

#### **5.7.4 Print**

The print section shows all parameters that can be printed as evaluation of the session.

The instructor can select which parameters are important for the evaluation according to their personal needs.

To change the setting, enable or disable the relevant checkmark.

In addition, the instructor can enable or disable the heading of the printout.

A personal logo can be added to the printout. The personal logo will be printed in the top left corner of the printout.

To upload a personal logo, click in the browse button and select the logo file.

The logo file needs to be in PNG format with a maximum size of 300x100pixels.

#### **5.7.5 Storage**

The print section shows how much data storage in the manikin has been used and how much is remaining.

The manikins can be set to use external data storage.

The external data storage functionality is based on the WebDAV protocol (Web-based Distributed Authoring and Versioning), which is enabled in the manikin. To use the service, a WebDAV server application has to be installed on the computer or service.

Note: It is highly recommended that only experienced IT personnel perform the installation and setup of the WebDAV server and configuration of the manikin. Incorrect installation and setup of the server may cause the manikin to malfunction.

If the manikin cannot be accessed after selecting external storage, the settings in the manikins, the setup or installation of the server is not correct. A hard reset, using the reset button as described in the manikin IFU, needs to be performed.

## **6 Viewing mode**

The viewing mode allows access to the manikin from another computing device, to view the actions performed.

To enter the viewing mode:

- Connect the computing device to the same network as the manikin.
- Open one of the web browsers described in Section 2.2
- Enter “http://Ambu.login” into the address field of the web browser.  
If the web browser does not accept “http://Ambu.login” type in the IP address “192.168.1.1”
- Click on “Viewing mode” button to start.
- Select the manikin you want to access.
- Select the graph type you want to view.

## 6.1 Graphical display in viewing mode

The viewing mode can be adjusted to show different graphs and displays.

For further information on how to change the display, see section 5.7.2.


## 6.2 Line graph:

The line graph display shows a continuous line diagram for the ventilations and compressions performed.

Ventilation is displayed in the top graph, while compression is displayed in the bottom graph.

If a ventilation with a stomach inflation or a compression with a wrong hand position is performed, an “X” will be displayed on the graphs.

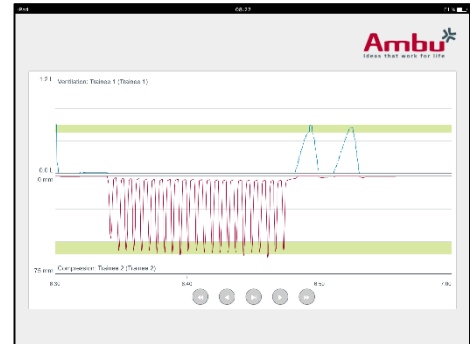To scroll through the graphs, the forward and backward buttons can be used.


## 6.3 Bar graph:

The bar graph display shows a continuous bar diagram for the ventilations and compressions performed.

Ventilation is displayed in the top graph, while compression is displayed in the bottom graph.

If a ventilation with a stomach inflation or a compression with a wrong hand position is performed, an “X” will be displayed on the graphs.

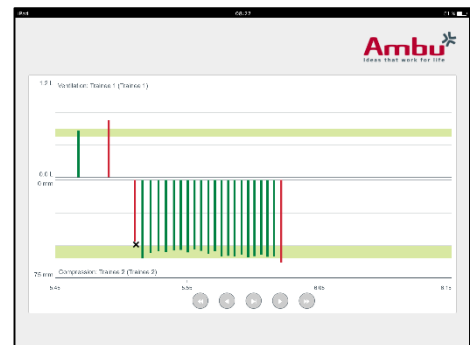To scroll through the graphs, the forward and backward buttons can be used.

## 6.4 Bubbles:

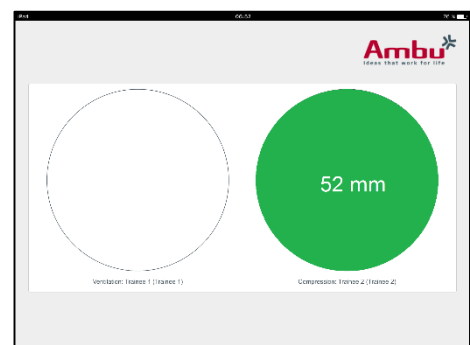
The Bubble display shows the ventilations and compressions as bubbles.

Ventilation is displayed on the left side, whilst compression is displayed on the right side.

The bubbles will turn green when the ventilations or compressions have been performed in accordance with the selected algorithm. If the performance is not in accordance with the selected algorithm, the bubbles will turn red.

The value inside the bubble show the performed compression/ventilation.

If a ventilation with a stomach inflation or compression with a wrong hand position is performed, an “X” will be displayed.


## 6.5 Patient Monitor:

The Patient monitor displays the performance of the trainees as a simulated patient monitor.

The top graph displays the selected ECG, the bottom displays a capnography curve.

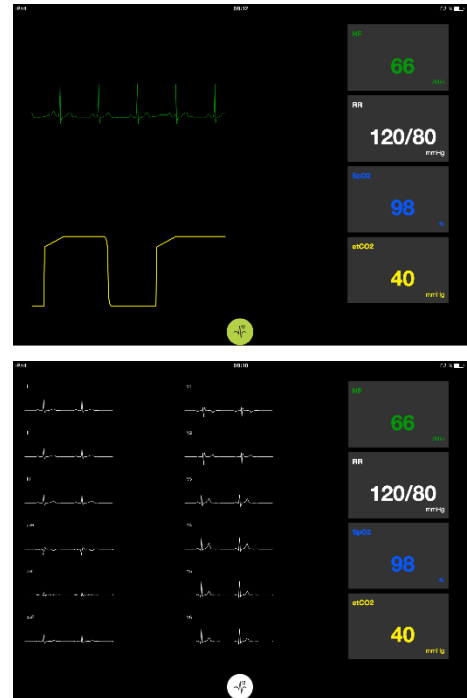To display a 12 lead ECG, click on the 12-lead ECG button.

## 7 Self-training

The self-training mode is used for instructor independent single trainee CPR training.

In self-training mode, the trainee can perform a CPR training and will get a result at the end of the session.

The display of the performance can be set to line graph, bar graph or bubble display.

To configure the self-training mode see section 8.4.

Note: The self-training mode is not available on the AmbuMan Advanced manikin model.

To enter the self-training mode:

- Connect the computing device to the same network as the manikin.
- Open one of the web browsers described in Section 2.2
- Enter “Http://Ambu.login” into the address field of the web browser.  
If the web browser does not accept “http://Ambu.login” type in the IP address “192.168.1.1”
- Click on “Self-training” button to start.

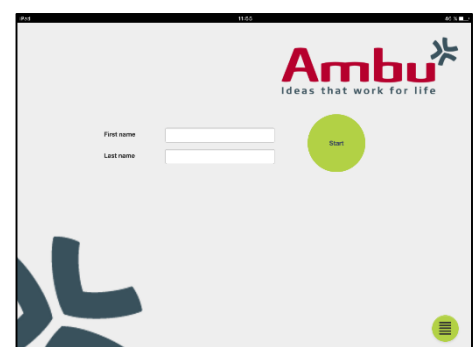
The Ambu Manikin Management Module now displays the self-training start screen.

The trainee has to enter their name.

To start the training click on “start”.

The trainee has now 10 second to go to the manikin.

The system will show a countdown until the session starts.

Once the session is finished, the Ambu Manikin Management Module will automatically display the training results.

The trainee can now review their performance.

When the session has finished, the trainee has the following options:

**Print:**

To print the results the trainee has to click on the print button.

**Save:**

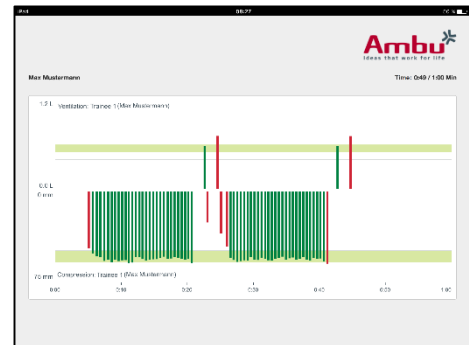
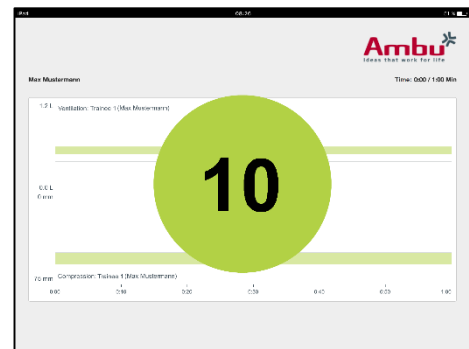
To save the data the trainee has to click on the save button.

**Delete:**

To exit the session the trainee has to click on the delete button.

A warning message will be displayed that has to be confirmed.

Note. If a session is deleted, no data can be restored.

## 8 Manikin Configuration

The general configuration of the manikin can be adapted to personal preferences.

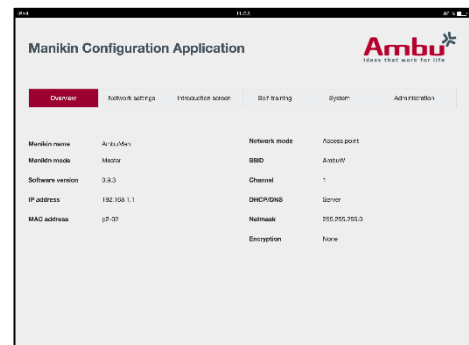
To enter the configuration follow the steps below:

1. Turn on the manikin
2. Connect your device to the manikin wireless (WLAN) or using a network cable.  
When using WLAN connect to the wireless network “AmbuW”. No password is required.
3. Open one of the web browsers described in Section 2.2
4. Enter “http://Ambu.login/config” into the address field of the web browser.  
If the web browser does not accept “http://Ambu.login/config” type in the IP address “192.168.1.1/config”

### 8.1 Overview

The overview shows the current network configuration of the manikin.

Further description on how perform changes to the settings are described in Section 8.2 – 8.6.

## 8.2 Network settings

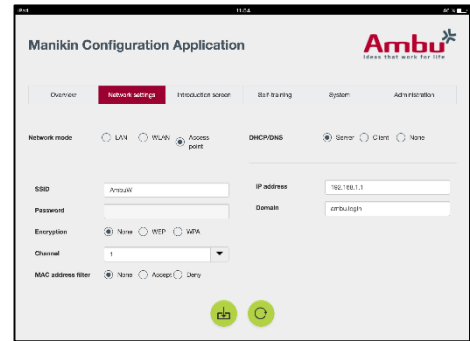
The manikins can be connected through LAN or WLAN, using the TCP/IP protocol.

It is recommended that only experienced IT personnel make changes to the network configuration.

Network mode:

Depending on the network infrastructure you want to use, the manikin can be configured for:

- *LAN*, if connected through a wired Ethernet cable
- *WLAN*, if connected to an existing WiFi network
- *Access Point*, if the manikin should create a wireless access point

Depending on the selected network mode, select DHCP/DNS:

- *Server*, if the manikin should assign the IP addresses
- *Client*, if the manikin should receive the IP address from an external DHCP server
- *None*, if the IP address will be assigned manually

If the manikin is set to WLAN or Access Point, the SSID can be configured according to the external network settings or personal preferences.

Note: If the *IP address* or *Domain* is changed, the domain name entered in the address field of the browser has to be adjusted accordingly, to access the manikin.

When finished, click on the “save” button to store the settings.

### 8.2.1 Single Manikin

When running a single-manikin setup the default settings do not need to be changed.

The manikin that is connected to the computing device needs to be set as “Master” manikin.

### 8.2.2 Multi-Manikin

When running a multi-manikin setup, the manikins need to be configured before the training.

The first manikin needs to be configured as the “Master” manikin. This manikin will be the manikin controlling the session and the data flow of all the other connected manikins. For further details on how to setup the Master, see section 8.2.

All the other manikins need the following parameters to be configured:

- *SSID*, needs to be the same as on the “Master” manikin, see section 8.2.
- *DHCP/DNS*, needs to be set as client, or needs to be assigned manually, see section 8.2.
- The manikins need to be set to “slave” manikins, see section 8.5.

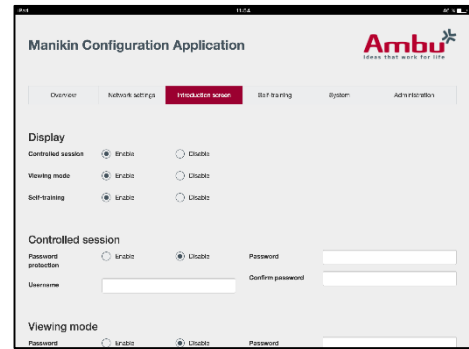
Note: If the SSID or the IP address range on the “Slave” manikins do not match the “Master” manikin, the manikins may not be able to connect and function correctly.

### 8.3 Introduction screen

The introduction screen section allows changing the availability of the different modes, or to set a password protection for the introduction screen.

To disable the availability of one of the views, click the relevant disable button.

If the controlled session, the viewing mode or the self-training shall be protected, a password can be set.


Note: If the password is forgotten, the manikin configuration cannot be accessed. A hard reset, using the reset button as described in the manikin IFU, needs to be performed.

When finished click on the “save” button to store the settings.

### 8.4 Self-training

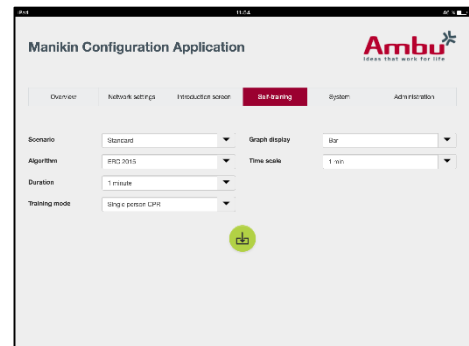The self-training section allows preparing the manikin for the self-training mode.

Select the required session parameters and click on the “save” button to store the settings.


### 8.5 System

In the system section, the following settings can be changed:

Manikin name:

The manikin name is the name that is displayed to identify the manikin being used.

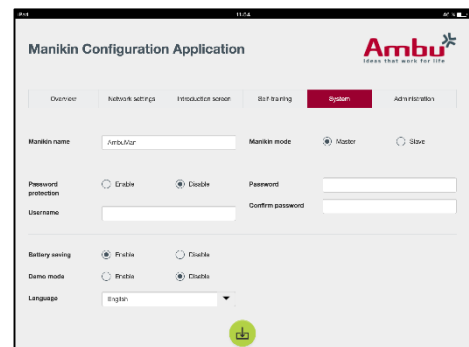The manikin mode can be adjusted to setup the manikin for a single-manikin or multi-manikin session. More details can be found in section 8.2.1 and 8.2.2.

Password protection:

If the manikin configuration shall be protected, a password can be set.

Note: If the password is forgotten, the manikin configuration cannot be accessed. A hard reset, using the reset button as described in the manikin IFU, needs to be performed.

Battery saving:

If the battery saving mode is enabled, the manikin will shut down after 30 minutes with no active session.

Language:

To change the language of the user interface, click on the dropdown button and select the required language.

If the required language is not available, a new language file can be uploaded into the manikin as described in section 8.6.

When finished, click on the “save” button to store the settings.

## 8.6 Administration

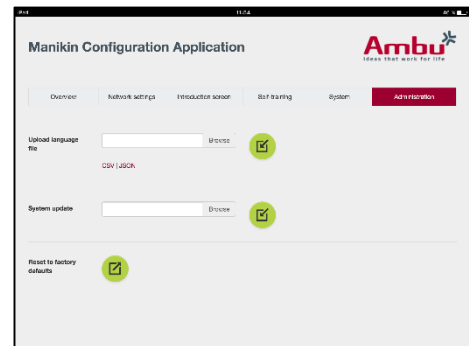
The administration section allows uploading new languages, performing a system update or a software reset.

To upload a new language file, click on “browse” and select the language file. Click on the “upload” button to upload the file to the manikin.

If a new language file shall be created, a template can be downloaded from the manikin.

To upload a system update file, click on “browse” and select the system update file. Click on the “upload” button to upload the file to the manikin.

To perform a software reset, click on the “reset” button.

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The screenshot displays the AmbuMan interface for creating a new course. The form includes fields for Date (2016-01-20), Time (11:58), Scenario (Default scenario), Algorithm (AMA 2010), Duration (20 seconds), Training mode (Two person CPM), and Instructor (Instructor 1). Below the form is a table for adding trainees:

No.	Trainee 1	Trainee 2	Manken	Go to	Delete
1	Trainee 1	Trainee 2	AmbuMan		

The interface also includes a sidebar with navigation icons and a top bar with 'AmbuMan' buttons and a play button.

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**Ambu A/S**