# Ambu® Endotracheal Tube Holder

## Single Patient Use. Fits Tube Sizes 3-9

**Ambu**  
Ideas that work for life

**Ambu**  
Ideas that work for life

**Corporate Head Office**  
**Ambu A/S**  
Batorbakken 13  
DK-2750 Ballerup  
Denmark

T +45 72 25 20 00  
F +45 72 25 20 50  
www.ambu.com

492 264031 - 08/2009 - Version 1.1 Printed in China

### DIRECTIONS FOR USE

#### Ambu® Endotracheal Tube Holder

Single Patient Use. Fits Tube Sizes 3-9

1. Slide ET Tube into the opening of the tube holder.
2. Advance bite block into the patient's mouth.
3. Rotate swing arm clockwise and engage it with ratcheted mechanism to hold ET Tube securely in place. Rotate swing arm in such a way that just enough pressure is applied against the ET Tube to hold. Do not over-tighten the swing arm.

- 4A. ET Tube Holder with Blue Strap: Attach yellow inducer (pull stick) spikes to the holes of the blue head strap (if not attached already). Slide blue strap with inducer underneath the patient's neck. Pull inducer from the other side of the neck. Slide the blue strap through the half moon opening and secure it to the spike of the tube holder.
- 4B. ET Tube Holder with White Velcro/Cloth Strap: Slide White strap with inducer underneath the patient's neck. Pull inducer from the other side of the neck. Slide the White Strap through the half moon opening. Secure the strap by attaching the velcro tab. Extra velcro tab is placed on the strap to secure smaller head patients or children. If required, simply peel the velcro tab and use it to secure the strap.

Make sure that head strap is not twisted. Adjust head strap for proper securing.

To release the ET Tube Holder

- A) with Blue Strap, simply remove the blue head strap from the stem hook or cut the strap.
- B) with White Strap, simply un-attach the velcro tab or cut the strap. Press ratcheted mechanism outward and rotate swing arm counter-clockwise to disengage. Remove ET Tube Holder and/or ET Tube per your local protocol.

#### WARNINGS

1. This device should only be used by qualified and authorized personnel. To be used only as directed by local EMS, hospital or other related authorities.

Caution: US Federal law restricts the device to sale by or on the order of a licensed practitioner.

2. For single patient intubations only. Re-use can result in crossinfection.
3. For short-term intubations only. Remove ET Tube holder after 24 hours.

4. Patient should be constantly monitored while the device is in use.

5. This device is to be functionally checked by qualified personnel prior to its use. Inspect all parts of the product and do not use the product if any parts are damaged. Ensure that the swing arm rotates freely and engages with the ratcheted mechanism easily.

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Ambu is a registered trademark of Ambu A/S Denmark.
Ambu S.A.R.L.